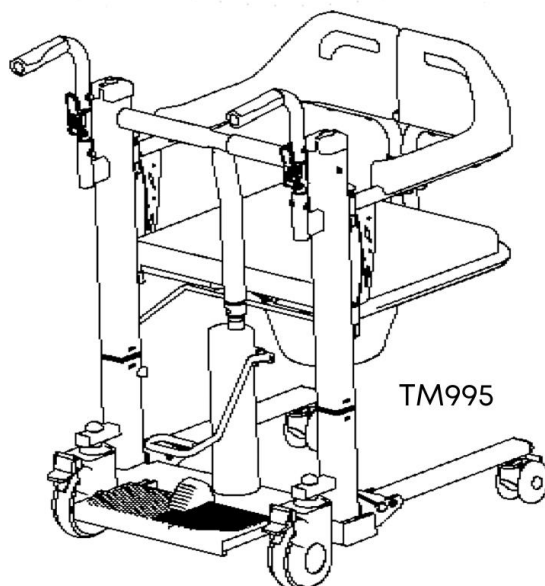
TheraMart

PT

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Cadeira Multifuncional TheraMove

ES

MANUAL DE INSTRUCCIONES

Silla Multifuncional TheraMove

EN

INSTRUCTION MANUAL

Multifunctional Chair TheraMove

[PT]

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[EN]

This instruction manual will present precautions and the safe use and operation of this chair.

- Before using this chair, carefully read this instruction manual to fully understand the safe and correct operational procedures.
- The correct operational procedures must not only be explained to the person using this chair but also to the caregiver team.
- After reading, store this instruction manual properly for future reference.
- If you have any questions, please contact TheraMart directly.

Version 01

Date: February 2025

ANVISA Registration: 82692690007

INDEX

1. Overview of the Instruction Manual –
2. Scope of Application –
3. Precautions for Safe Use –
4. Product Specifications and Component Names –
5. Performance Features –
6. Usage Restrictions –
7. Installation Instructions –
8. Usage Instructions –
9. Maintenance and Care Methods –
10. Maintenance Requirements –
11. Packaging List –
12. Product Transportation and Storage Conditions –
13. After-Sales Service –
14. Additional Information –

1. Overview of the Instruction Manual

Please read the user manual carefully before using the product.

This manual provides instructions on operation, assembly methods, and basic troubleshooting.

This manual includes maintenance tips for the multifunctional chair and troubleshooting guidelines. Please keep it in an accessible location and refer to it as needed.

The notes and illustrations in this manual may slightly differ from your actual model, as we continuously improve and update our components and designs.

If you are unsure about any aspect of the operation, please contact your dealer for clarification.

2. Scope of Application

The device is designed to assist in the safe and efficient transfer of people with reduced mobility, such as people with disabilities, the elderly, and patients with conditions that limit their movement, between different surfaces such as chairs, beds, sofas, or other furniture.

Components and accessories:

- Adjustable seat:

Provides comfort and adapts to different furniture heights.

- Hydraulic system:

Allows smooth lifting and adjustment, reducing operator effort.

- Wheels with locks:

Ensure stability during use.

- Safety belt:

Ensures user protection during transfer.

- Robust steel frame:

Provides durability and resistance to the equipment.

3. Precautions for Safe Use

Notice

- Make sure to read these “Precautions for Safe Use” carefully before using to use it properly.
- If you do not follow the instructions in the “Precautions for Safe Use,” the product may be damaged, people may be injured, and even life may be at risk.
- The operator must understand the usage instructions in the instruction manual before use.
- Do not allow children under 12 years old or people who do not understand the operation to use the chair.
- Do not use the chair without a caregiver, as injuries may occur.

All the above items are important for safety, so be sure to comply with them.

* After reading the instruction manual, keep it where it is always visible.

Warning

- Before use, adjust the height to the appropriate position and make sure the seat panels on both sides are level.
- When entering or exiting the transfer vehicle, avoid leaning on the frame, as this may cause tipping.
- Before using the lift vehicle, ensure that the safety buckle and seat belt are properly fastened.
- Users must ensure the wheels are locked before entering or exiting the lift.
- When reaching for objects outside the transfer vehicle (in front, to the sides, or behind), avoid excessively leaning your body to avoid compromising the stability of the vehicle and prevent tipping.
- Do not bend your knees to reach items on the floor.
- During seat height adjustment, the front wheels must be locked. The operator must control the height by operating the hydraulic pump with their foot.
- Do not use inertia to climb steps with the forklift, as this may cause accidents or damage. When going up or down stairs, the

operator must not move the user together with the forklift, as this poses a safety risk to the user.

- The transfer vehicle should only be used on flat surfaces. Be aware of holes and obstacles to avoid the risk of tipping.
- On slopes with an inclination greater than 5 degrees, the operator must guide the transfer vehicle in reverse, descending carefully. If descending forward, the user may lean forward and fall.
- On sloped terrain (with an inclination less than 10 degrees), all four wheels must be locked to ensure safety.
- The brakes on all four wheels of the transfer vehicle should only be used for parking. They are not designed for deceleration or braking during movement.
- The operator must control the hydraulic pump using only their feet. Avoid stepping on the control pedal without proper support, as this may damage the pump and compromise safety.



4. Product Specifications and Component Names

Model/Specification Parameter	TM995(3)/460mm
Length	770±10mm
Width	590±10mm
Height	930-1190±10mm
Front wheel size	5 polegadas
Rear wheel size	3 polegadas
Seat width	460±10mm
Seat depth	400±10mm
Seat height from ground	460-720±10mm
Net weight	32kg
Maximum load capacity	130kg

Note:

We reserve the right to make technical and appearance changes to this product.

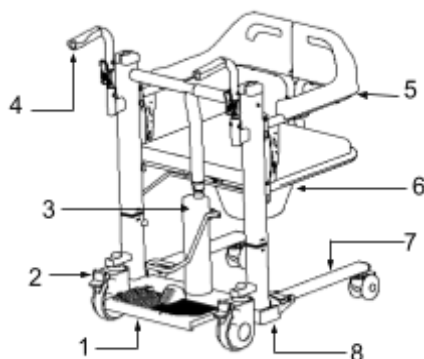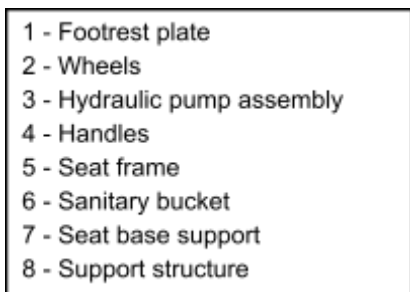This product consists of a support, casters, pedal, base legs, hydraulic pump assembly, seat structure, etc. The material is welded with high-strength steel tubes. As shown in Figure 1.

* The diagrams in this manual are for user reference only.
Please refer to the actual product for real use.

5. Performance Features

- It is convenient for people with mobility difficulties to transfer from wheelchairs to sofas, beds, seats, etc.
- The wide and practical design for opening and closing facilitates the operator in providing support to the user from the lower part, reducing effort and preventing possible lower back injuries to the operator.
- Load capacity: Supports up to 130 kg, adapting to people of different body types with safety and stability.
- Adjustable height: The seat features continuous height adjustment, making it compatible with furniture and installations of various heights.

- Hydraulic lift system: The hydraulic pump, operated by foot control, allows smooth and precise adjustments, ensuring horizontal elevation and reducing operator effort.
- Silent medical casters: Equipped with high-quality silent wheels and hub brakes, ensuring stability and reliability during use.
- Reinforced safety: The design includes double protection, including a safety buckle and a front handrail, providing greater security for the user.
- Comfort: Seat with a thick special cushion, offering comfort even during long periods of use. The cushion cover is removable and washable, making cleaning easier.
- Practical structure: The base includes a quick-release system, and the legs are foldable, allowing compact storage and easy transport.


6. Usage Restrictions

The multifunctional chair is designed exclusively for indoor use, with a maximum operating speed of ≤ 3 km/h, ensuring safety and stability during handling.


7. Installation Instructions

1. Remove the main structure of the multifunctional chair and the two 5-inch directional front wheels. Place the main structure on a flat surface, align the 5-inch front wheels with the chassis mounting holes, and insert them completely. Insert the M12 iron axle and the M12 locking nut, respectively, and use the corresponding M19 multifunctional tool. After tightening the nut with a wrench, reattach the M12 nut cap. (As illustrated in Figure 2)

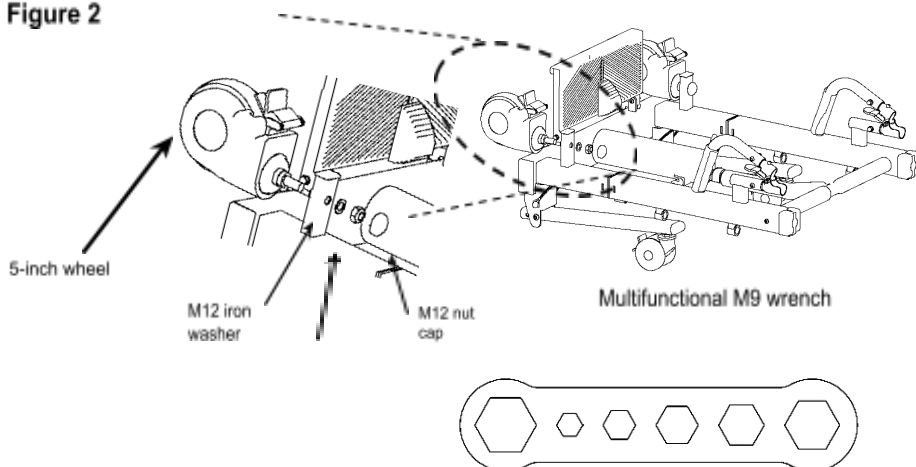Figure 2

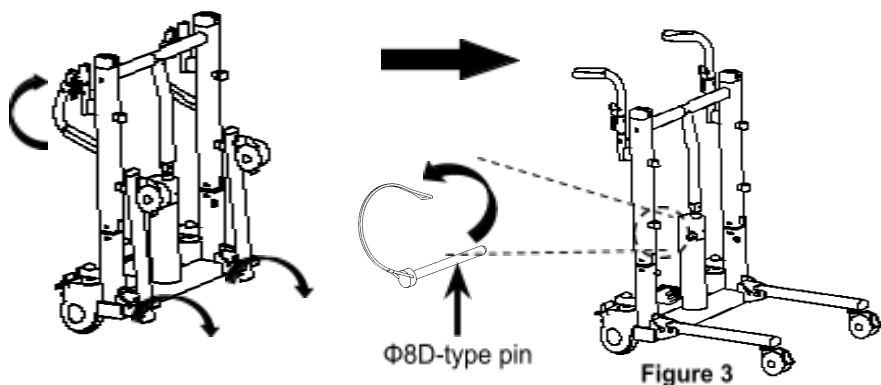
2. Straighten the main structure of the multifunctional chair. Pull the two $\Phi 8D$ -type locking pins located at the base, and then adjust the base legs on both sides by moving them downward to the appropriate position. Reinsert the locking pins to securely fix them. Then, lift the side handles into the usage position. (As illustrated in Figure 3)

3. Position the hydraulic pump control pedal into the designated hole of the hydraulic pump. Use the M4 hex wrench provided with the multifunctional chair to firmly tighten the M8 headless

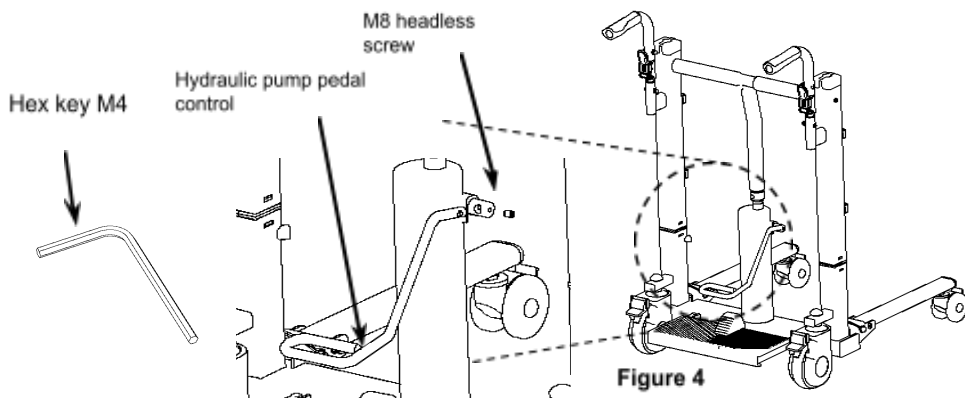screws on the hydraulic pump. Ensure that the screws are properly tightened to avoid any movement or oscillation of the pedal during use. (As illustrated in Figure 4)

4. To install the seat frame, position the swivel sleeve of the right seat frame directly above the swivel sleeve of the right column tube of the support. Carefully align the holes and press the quick-release axle switch to fix the right seat frame in place, as shown in Figure 5. After installation, check whether the seat frame is well adjusted, with no visible gaps, and that it rotates smoothly and flexibly. The left seat frame can be installed using the same procedure.

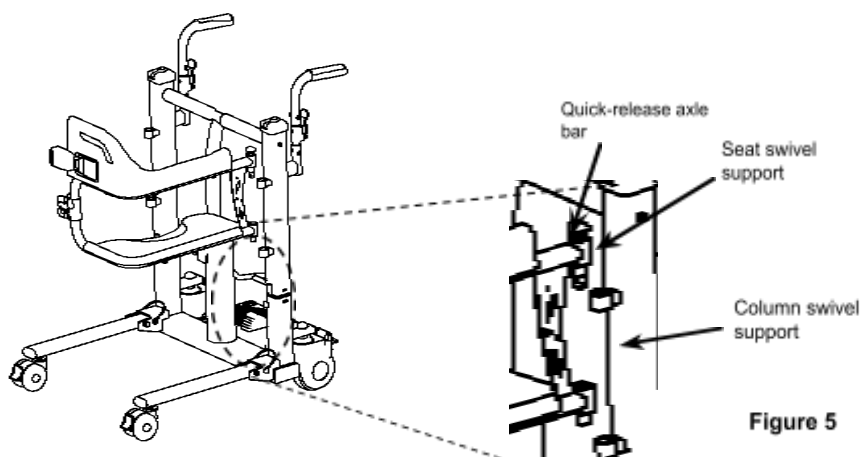
Figure 5

5. To use the sanitary bucket, first install the metal support structure into the mounting holes located on both base columns. Ensure that the support is well positioned and securely fixed. Then, fit the sanitary bucket into the support from the rear side, making sure it is properly adjusted, as shown in Figure 6.


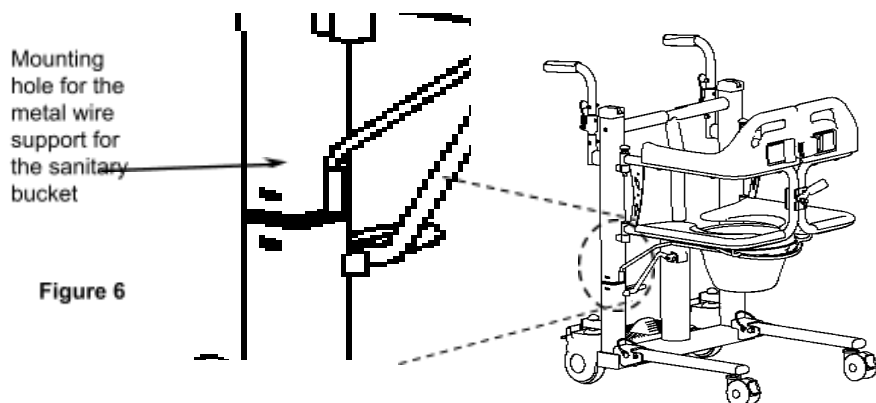
Figure 6

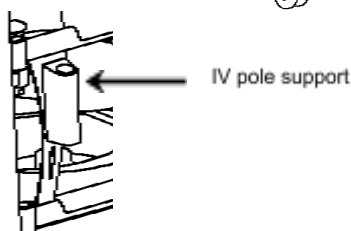★ If you choose to install the IV pole, you can choose to install it on the left or right side of the seat support, as shown in Figure 7.

8. Instructions for Use

* As an example, consider the operator transferring the user from the seat to the bed. The operational steps are explained. The transfer method between other furniture and installations is the same. (The human body model is for reference only, use the actual product for real operation.)

- Have the user sit upright, and the operator moves the transfer vehicle and adjusts the height of the seat frame of the transfer vehicle so that the seat frame is slightly higher than the seating surface.
- After opening the seat frames on both sides, move the transfer vehicle directly in front of the user. Adjust the position of the transfer vehicle so that the user's center of gravity is aligned with the center of the seat frame, and the lower part of the seat frame lightly touches the surface of the chair. (Figure 8)
- Lock the 5-inch front wheel brakes, lift the user's feet, and place them on the footrests.
- The user may lean slightly to the left and right while the operator rotates the corresponding side frame, allowing the user to sit on the left and right seat frames of the transfer vehicle in sequence. (Figure 9)
- When the user is fully seated on the seat frame, the left and right seat frames are fully closed, the safety lock engages automatically, and the operator fastens the safety belt.

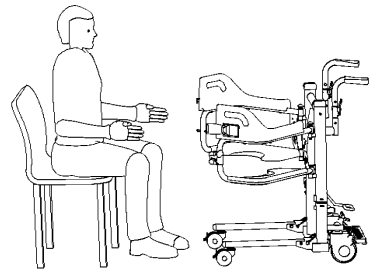
Figure 8



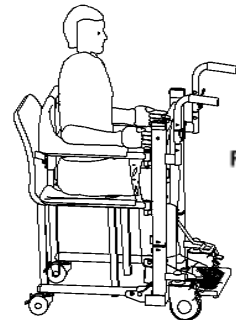
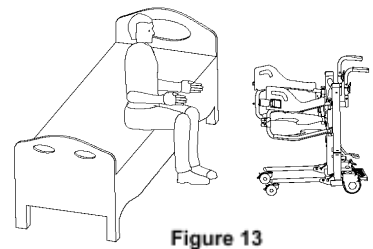
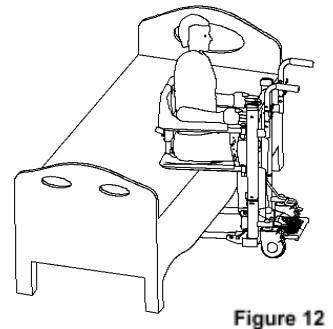
Figure 9

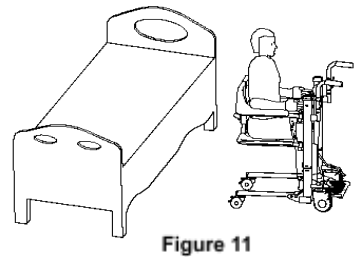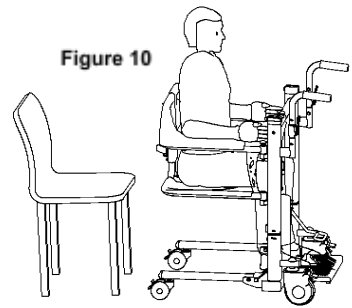
- The user adjusts to a comfortable seated position with hands naturally resting on the armrests.
- The operator checks the distance between the bottom surface of the transfer vehicle seat frame and the chair, raises the seat frame by 2 to 3 centimeters, releases the 5-inch front wheel brake, and slowly pulls the transfer vehicle away from the seat. (Figure 10)
- Push the chair to the edge of the bed, holding the front handle. Check the seat frame height relative to the bed edge to ensure the bottom of the seat frame is slightly higher than the bed surface. If height adjustment is necessary, lock the front wheel brakes before adjusting. (Figure 11)
- Push the chair completely into the desired position on the bed. Lock the 5-inch front wheel brakes and readjust the height of the seat frame so that its bottom surface lightly touches the surface of the bed. (Figure 12)
- Unfasten the safety belt and unlock the safety latch located behind the backrest. Slowly open the left and right seat frames, allowing the user to gently lower the buttocks onto the bed. (Figure 13)
- Unlock the 5-inch front wheels, lift the user's feet from the footrests, and move the chair away from the bed.


Attention: The hydraulic pump pedal control must be used together with the Height Control.

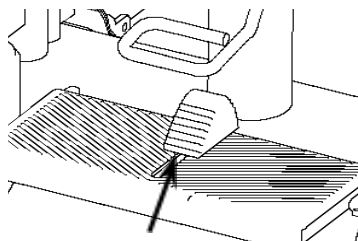- When the Height Control is in the inner position, repeatedly press the pedal to control lifting. (See Figure 14)
- When the Height Control is in the outer position, press the pedal all the way down to control lowering. (See Figure 15)

Limit control block facing inward

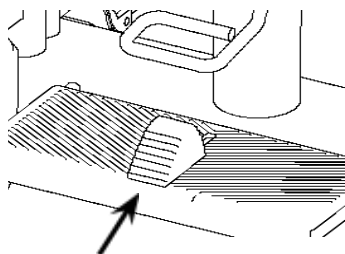Figure 14

Limit control block facing outward

Figure 15

Advance adjustment of the hydraulic pump height, controlled by pin. This operation must be performed in the absence of users. It is not possible to adjust when a person is using it.

(As shown in Figure 16)

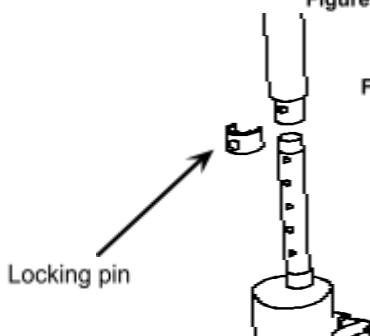
Figure 16

9. Maintenance and Care Methods

- Avoid scratching or scraping the product's surface with hard objects to prevent damage to the material.

- For daily cleaning and maintenance, use a neutral detergent. Wipe the surface with a damp cloth, then dry with a clean, dry cloth.
- Do not expose the product to strong acidic or alkaline substances to avoid corrosion.

10. Maintenance Requirements

Frequency:

Daily: Check general cleanliness and operation of the hydraulic system.

Monthly: Inspect wheels, locks, and hydraulic components for signs of wear.

Annually: Full inspection by a qualified technician, including lubrication of the hydraulic system and replacement of worn parts.

Responsible Party:

Basic maintenance by the user; technical inspections by authorized distributor.

Service life: The product has a normal service life of 3 years from the manufacturing date.

11. Packing List

The product is delivered with the following items:

- **Main structure:** Composed of the support, seat, and movement components.
- **Padded seat:** 1 unit
- **Side supports and backrests:** 1 set (left and right)
- **Front wheels (with locks):** 2 units
- **Rear casters:** 2 units
- **Safety belt:** 1 unit
- **Assembly tools and accessories:** 1 set
- **Instruction manual:** 1 unit

12. Product Transportation and Storage Conditions

Transport: Products must be handled with care during transportation. Throwing, rolling, or applying excessive pressure is not allowed.

Storage:

The product must be placed in a dry and ventilated area. Avoid environments with high temperatures or sudden temperature changes. The product must be kept away from chemically corrosive substances such as acids and alkalis.

Environmental Restrictions for Transport and Storage:

- **Temperature:** -10°C to +55°C
- **Relative Humidity:** 25% to 93% (non-condensing)

Requirements: Dry and ventilated place, protected from direct sunlight and corrosive agents. Avoid locations with abrupt temperature changes.

13. After-Sales Service

Product Warranty List

Part Name		Warranty Coverage	Warranty Period
Main frame	Welded components, such as base structure, lift structure, seat structure, wheel frame, etc.	Bent, deformed, or broken due to external force	No warranty
		Welded parts fail or break during normal use	36 months
		Product surface shows excessive rust or areas painted with spray are peeling or blistering	12 months
Electronic control part		Private modifications, disassembly, damage, etc.	No warranty
		Out of control	12 months
		Cruise range is reduced by	90 days

	less than 20% (inclusive)	
	Opened water part in particular	No warranty
	Fuse cannot be recharged	12 months
	Fuse	No warranty
	Water intrusion or human-caused damage	No warranty
	Normal use of the entire machine (except fuse)	12 months

Repairs must be performed by trained professionals.

Warranty Policy

Warranty Coverage:

The main frame (product structure) has a three-year warranty.

Consumable parts not covered by the warranty: plastic casing, rubber armrest pads, casters, decorative parts, and seat backrests.

Repair Requests:

If the product shows quality issues within one year from the date of purchase, the user may contact the company's after-sales service department, office, or authorized dealer.

An invoice and warranty card must be presented to request repairs, and parts will be provided at a reasonable cost.

If the user cannot provide the invoice, the warranty period will be calculated based on the lot number or manufacturing date, with a one-month extension.

Situations Not Covered by Warranty:

- Failures resulting from unauthorized disassembly, repairs, or modifications.
- Damage caused by accidental drops during use or transportation.

- Malfunction due to improper use or failure to follow the instructions in the manual.
- Damage resulting from unforeseen natural disasters such as fire, earthquakes, or floods.

Note: To preserve warranty coverage, strictly follow the usage and maintenance instructions described in the product manual.

14. Additional Information

Symbols related to this product's safety requirements and their meanings

			
Keep away from rain	Fragile, Handle with care	Caution! Please read the instructions	Stacking layer limit

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